

tcs[®]
Flexible Partial



Guaranteed Unbreakable

TCS[®] UNBREAKABLE (aka TCS) *nylon thermoplastic*

- Guaranteed unbreakable
- Lightweight yet durable
- Ideal degree of flexibility
- Stain resistant
- Repairable and rebasable
- Biocompatible
- Millions of successful cases prescribed

SPECIFY TCS ON YOUR LAB SLIP



AVAILABLE IN 5 TRANSLUCENT SHADES



SHADE 1
LIGHT PINK



SHADE 2
STND PINK



SHADE 3
LT/DK PINK



SHADE 4
DARK PINK



NATURAL
FOR CLASPS

RECOMMENDED INSURANCE CODES

D5225 **Upper**

D5226 **Lower**

WITH EVERY TCS PARTIAL YOU PRESCRIBE, RECEIVE:

- 1-week sample of tcs® Concentrated Dental Appliance Cleaner
- TCS patient care instructions



indications

- Economical, aesthetic solution for missing teeth
- Concerns over metal restorations
- Gingival veneers to mask exposed root surfaces
- Allergies to monomer and nickel
- Ideal as a transitional restoration during the healing period for implants
- Presence of tori, oral cancer, cleft palate

Tips

1. For best results use lightweight alginate impression materials.
2. When making adjustment with a hand piece, set at low (20,000 to 25,000 rpm) and use rapid back and forth scraping movements. Do not use a lot of pressure nor leave the bur in one spot. TCS recommends using green mounted points and rubber points for minor adjustments.
3. Soak the partial in very hot clean tap water for about 1 minute before initial insertion. When case cools to a tolerable temperature gently insert it into patient's mouth.
4. Recommend tcs® Appliance Cleaner to your patients for optimal maintenance.
5. Use the TCS Animation Video (1.5 min.) & Health Briefs with Terry Bradshaw (5 min.) for patient chairside education.